


## Algorithm: identifying and managing phosphorus overload in cats diagnosed with CKD

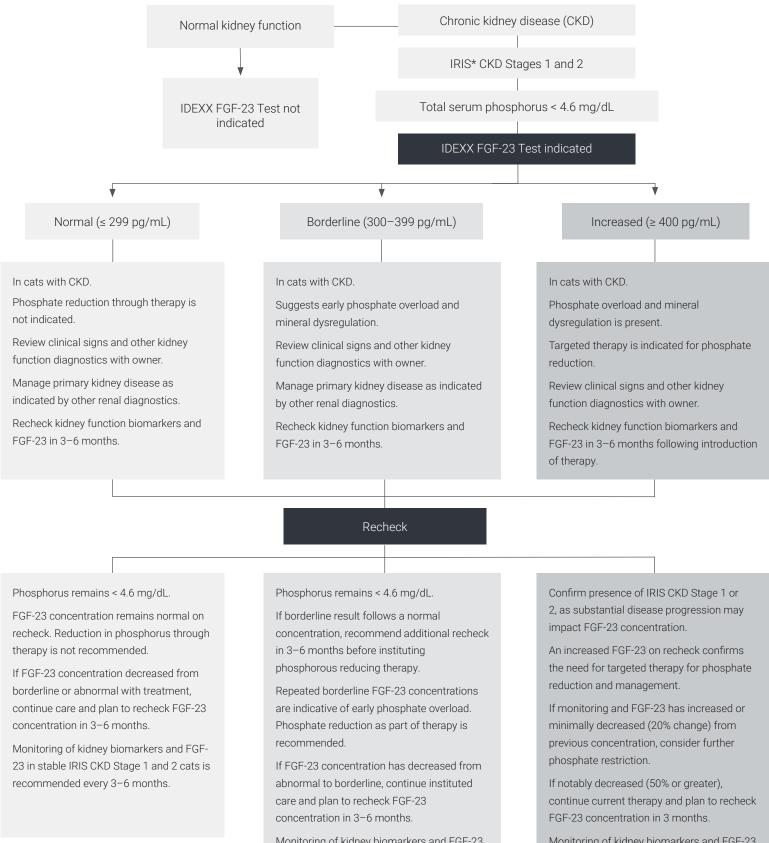

Monitoring of kidney biomarkers and FGF-23 in stable IRIS CKD Stages 1 and 2 cats is recommended every 3–6 months. Monitoring of kidney biomarkers and FGF-23 in stable IRIS CKD Stages 1 and 2 cats is recommended every 3–6 months.

IDEXX

The IDEXX FGF-23 Test provides an evidence-based approach to recognize phosphorus overload earlier<sup>1-4</sup> and recommend therapy to reduce phosphorus intake, supporting management of chronic kidney disease (CKD) in cats.

The IDEXX FGF-23 Test should only be run in cats with diagnosed or highly suspect IRIS CKD Stages 1 and 2. It is not recommended for cats with uncontrolled hyperthyroidism, profound anemia, or systemic inflammation.

**Please note**: When making changes to therapy impacting phosphorus intake or absorption, waiting at least 2 months to recheck FGF-23 is recommended.

## References

- 1. Finch NC, Geddes RF, Syme HM, Elliott J. Fibroblast growth factor 23 (FGF-23) concentrations in cats with early nonazotemic chronic kidney disease (CKD) and in healthy geriatric cats. J Vet Intern Med. 2013;27(2):227–233. doi:10.1111/jvim.12036
- 2. Geddes RF, Elliott J, Syme HM. Relationship between plasma fibroblast growth factor-23 concentration and survival time in cats with chronic kidney disease. J Vet Intern Med. 2015;29(6):1494–1501. doi:10.1111/jvim.13625
- 3. Geddes RF, Finch NC, Elliott J, Syme HM. Fibroblast growth factor 23 in feline chronic kidney disease. J Vet Intern Med. 2013;27(2):234–241. doi:10.1111 /jvim.12044
- 4. Seiler S, Heine GH, Fliser D. Clinical relevance of FGF-23 in chronic kidney disease. Kidney Int Suppl. 2009;76(114):S34–S42. doi:10.1038/ki.2009.405

\*IRIS is the International Renal Interest Society

© 2022 IDEXX Laboratories, Inc. All rights reserved. • 09-2690237-00\_EN-US • The IDEXX Privacy Policy is available at idexx.com.