SNAP* Cortisol Test Kit

For use only with the IDEXX SNAP* Reader Analyzer and the IDEXX SNAPshot Dx* Analyzer.

For veterinary use only.

[English version](#)

Overview

The SNAP Cortisol Test Kit is an enzyme-linked immunosorbent assay (ELISA) for the quantitative measurement of cortisol in dog serum. For in vitro use only.

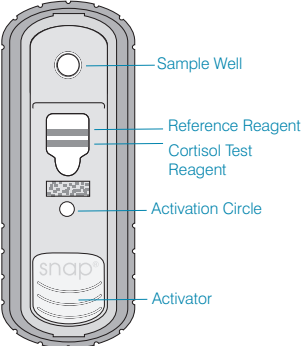
SNAP Cortisol test ranges

Dynamic ranges:

- 2.5–30 µg/dL (69–828 nmol/L) for post-ACTH stimulation, Cushing’s-suspected dogs and therapeutic monitoring > 10 µg/dL (>276 nmol/L)
- 0.5–10 µg/dL (14–276 nmol/L) for all other tests

Precautions and warnings

- Run the SNAP calibration device at the beginning of each day (SNAP Reader analyzer only).
- This test cannot be visually interpreted. The IDEXX SNAP Reader analyzer or SNAPshot Dx analyzer must read all tests.
- Use a separate pipette tip and sample tube for each test.
- Do not use components after their expiration dates.
- Do not mix components from different lots of test kits.
- Do not use a device that does not have two blue stripes visible before adding the sample.
- Keep the device in its sealed foil pack at room temperature (18–27°C/64–80°F) for at least 10 minutes before use.
- Prewarming of the conjugate solution is not required.
- Keep the SNAP device horizontal during the entire test procedure.**
- IMPORTANT: Do not depress the activator until instructed.**
- Insert the device into the analyzer immediately after activation.
- WARNING:** Conjugate – H316/P332+P313/EUH208. Causes mild skin irritation. If skin irritation occurs: Get medical advice/attention. Contains Proclin. May produce an allergic reaction.
- IMPORTANT:** Do NOT use the SNAP Cortisol test if the patient has received:
 - Dexamethasone (doses >0.1 mg/kg) in the last 7 days.
 - OR**
 - Prednisone or other corticosteroids in the past 4 weeks.

Storage

- SNAP devices and test reagents must be stored at 2–7°C (36–45°F).
- Do not heat or microwave the kit components.
- Take out only the kit components you will need.
- Do not leave kit components at room temperature for more than 8 consecutive hours.

Kit components

- Conjugate solution
- SNAP test devices
- Disposable pipette tips for 300 µL manual pipettor
- Disposable pipette tips for the 25 µL and 100 µL manual pipettors (for use with the SNAPshot Dx analyzer only)
- Disposable sample tubes
- Workstation

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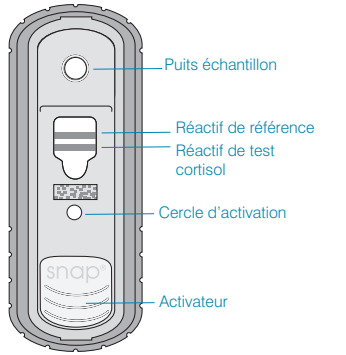
Componentes required (not provided in kit)

SNAP Reader analyzer users:

- VerTest* pipette tips
- 300-µL manual pipettor (provided with the SNAP Reader analyzer)

SNAPshot Dx analyzer users:

The following required components are provided with the SNAPshot Dx analyzer:

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Sample information

- Do not use plasma or whole blood. Use serum, either fresh, previously frozen or stored at 2–7°C (36–45°F) for up to 7 days. To store longer than 7 days, freeze the sample (-20°C/4°F or below). Remove serum from the red blood cells before freezing. Bring previously refrigerated or frozen samples to room temperature 18–27°C (64–80°F) before running assays.
- SNAP cortisol sample size requirements:
 - 25 µL for post-ACTH stimulation, Cushing’s-suspected dogs and therapeutic monitoring > 10 µg/dL (>276 nmol/L)
 - 100 µL for all other tests
- Centrifuge previously frozen or older samples before use.
- Results of this test are not affected by hemolysis or lipemia.

Test procedure

SNAP Reader analyzer users:

The VerTest analyzer provides on-screen instructions throughout the analysis. For complete instructions, refer to the Running Quantitative SNAP Tests chapter in the SNAP Reader Operator’s Manual.

SNAPshot Dx analyzer users:

The SNAPshot Dx analyzer provides on-screen instructions throughout the analysis. For complete instructions, refer to the Using the SNAPshot Dx Analyzer chapter in the SNAPshot Dx Operator’s Manual.

| Test | Sample Volume | Conjugate Volume |
|--|---------------|------------------|
| Baseline Cortisol | 100 µL | 300 µL |
| Post-Dexamethasone Cortisol (post low- or high-dose dexamethasone) | 100 µL | 300 µL |
| Post-ACTH Stim Cortisol: Cushing’s Suspected | 25 µL | 300 µL |
| Post-ACTH Stim Cortisol: Addison’s Suspected | 100 µL | 300 µL |
| Post-ACTH Stim Cortisol: Therapeutic Monitoring | 100 µL | 300 µL |
| Post-ACTH Stim Cortisol: Therapeutic Monitoring >10 µg/dL | 25 µL | 300 µL |

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Interpreting test results

The recommendations below are intended to provide general guidance only. As with any diagnosis, you should use clinical discretion with each patient based on a complete evaluation of the patient, including history, physical examination and complete laboratory profile.

NOTE: Dexamethasone (doses >0.1mg/kg), prednisone and other corticosteroids are expected to artificially increase the reported result.

A. ACTH stimulation test

| Pre-ACTH | Interpretation | Post-ACTH | Interpretation |
|-----------|---|-------------|---|
| <2 µg/dL | Must evaluate in conjunction with post-ACTH result. If both results are <2 µg/dL, results are consistent with hypoadrenocorticism. Begin treatment with mineralocorticoid and/or glucocorticoid as appropriate. | <2 µg/dL | Ideally, should be evaluated in conjunction with pre-ACTH result. If both results are <2 µg/dL, results are consistent with hypoadrenocorticism. Begin treatment with mineralocorticoid and/or glucocorticoid as appropriate. |
| 2–6 µg/dL | Normal | 2–6 µg/dL | Inconclusive |
| >22 µg/dL | Consistent with Cushing’s syndrome; perform high-dose dexamethasone suppression. | 6–18 µg/dL | Normal |
| | | 18–22 µg/dL | Equivocal; Cushing’s syndrome possible |
| | | >22 µg/dL | Consistent with Cushing’s syndrome; perform high-dose dexamethasone suppression to discriminate between PDH and ATH, ACTH level and/or abdominal ultrasound. |

B. Low-dose dexamethasone suppression test

| 4-hour cortisol level | 8-hour cortisol level | Interpretation |
|---------------------------------|---------------------------------|---|
| Not necessary | <1 µg/dL | Normal |
| 1–1.5 µg/dL | 1–1.5 µg/dL | Inconclusive, consider repeating in 8–12 weeks |
| >1.5 µg/dL and >50% of baseline | >1.5 µg/dL and >50% of baseline | Consistent with Cushing’s syndrome; perform high-dose dexamethasone suppression, endogenous ACTH concentration and/or abdominal ultrasound to discriminate between PDH and ATH. |
| <1.5 µg/dL or <50% of baseline | >1.5 µg/dL and >50% of baseline | Consistent with PDH |
| >1.5 µg/dL or >50% of baseline | >1.5 µg/dL and <50% of baseline | Consistent with PDH |
| <1.5 µg/dL or <50% of baseline | >1.5 µg/dL and <50% of baseline | Consistent with PDH |

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© 2017 IDEXX Laboratories, Inc. Alle Rechte vorbehalten.
© 2017 IDEXX Laboratories, Inc. Tutti i diritti sono riservati.