<p><b>Canine Heartworm Antigen Test Kit</b></p>	
<p>For veterinary use only</p>	
<p>English Version</p>	

#### PetChek® HTWM PF

The PetChek Canine Heartworm Antigen Test is an enzyme immunoassay for the detection of *Dirofilaria immitis* (*D. immitis*) antigen in canine or feline serum or plasma. A microtritation format has been devised in which antibodies to *D. immitis* antigen are coated on test wells. Upon incubation of the sample in the coated test well, antigen in the sample forms complexes with the coated antibodies.

Following removal of the sample, an enzyme conjugated antibody is added and binds to antigen captured in the well. In the final step of the assay, unbound conjugate is washed away and enzyme substrate/ chromogen is added. Subsequent color development indicates the presence of heartworm antigen in the sample.

#### Precautions and Warnings

- Do not expose TMB Substrate Solution to direct sunlight or any oxidizing agents.
- Care should be taken to prevent contamination of kit components.
- Optimal results will be obtained by strict adherence to this procedure. Careful pipetting and washing technique throughout this procedure are necessary to maintain precision and accuracy.

#### Storage

Store all reagents at 2–8°C. Allow reagents to come to room temperature (18–25°C) before use. Precipitation may occur in Stop Solution. Bring to room temperature (18–25°C) and mix by swirling before use.

#### Kit Components

- |  |        |
|--|--------|
| 1 Anti-Heartworm Antibody Coated Plates  | 2 x 96 |
| 2 1 bottle Anti-Heartworm: HRPO Conjugate (horseradish peroxidase), (preserved with gentamicin and Kathon) | 20 mL  |
| 3 1 bottle HTWM Positive Control: <i>D. immitis</i> antigen positive serum (preserved with Kathon)         | 4 mL   |
| 4 1 bottle HTWM Negative Control: serum free of <i>D. immitis</i> antigen (preserved with Kathon)          | 4 mL   |
| 5 1 bottle TMB Substrate Solution  | 25 mL  |
| 7 1 bottle Wash Concentrate (10X) (preserved with gentamicin)  | 100 mL |
| 8 1 bottle Stop Solution   | 25 mL  |
| • Precision pipette capable of dispensing 0.1 mL (100 µL) of sample  |        |
| • Disposable pipette tips  |        |
| • Well Holder  |        |

#### Materials Required but Not Provided

- Distilled or deionized water
- Wash bottle

#### Use of precision pipette to dispense 100 µL

Place disposable pipette tip on pipette. Depress button on pipette, place tip in sample and release button slowly to pull up sample. Depress button to dispense sample in well.

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<p><b>Preparation of Wash Solution</b></p>	
<p>Dilute Wash Concentrate ten-fold (1/10) with distilled/deionized water (e.g. 5 mL of Wash Concentrate plus 45 mL of water for each 12 wells). Salt crystals may form in the Wash Concentrate upon storage. If this occurs, allow concentrate to come to room temperature (18–25°C) and mix by swirling to redissolve crystals before preparing wash solution.</p>	

<p><b>Sample Information</b></p>	
<p>Serum or plasma may be used in this test. Serum or plasma (e.g., Heparin, EDTA) may be stored for up to 7 days at 2–8°C. Highly hemolyzed samples should not be used, however moderately hemolyzed or lipemic samples will not affect results. For longer storage, sample should be frozen (-20°C or colder).</p>	

<p><b>Test Procedure</b></p>	
<p>Allow reagents to come to room temperature (18–25°C) before use. Mix reagents by gentle swirling.</p>	

- Count the number of samples to be tested plus two wells for the positive and negative controls (one each). If using the well holder, remove the required number of wells from the bag and place into the well holder (up to 2 strips or 24 wells). If using the plate frame, remove the required number of complete strips from the bag and place into the 96-well plate frame. Record the positions of samples and controls on a worksheet.

<p><b>1</b></p>	
-----------------	---

<p><b>2</b></p>	
-----------------	---

<p><b>3</b></p>	
-----------------	---

- Using the precision pipette provided in kit and a separate pipette tip for each sample, add 100 µL of serum or plasma to appropriate wells. Incubate sample in well for 5 minutes at room temperature (18–25°C).

<p><b>4</b></p>	
-----------------	---

<p><b>5</b></p>	
-----------------	---

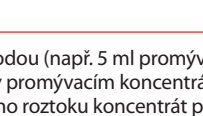
<p><b>6</b></p>	
-----------------	---

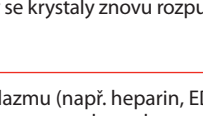
<p><b>7</b></p>	
-----------------	---

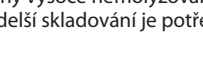
<p><b>8</b></p>	
-----------------	---

<p><b>9</b></p>	
-----------------	---

<p><b>10</b></p>	
------------------	---

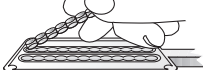
<p><b>11</b></p>	
------------------	---

<p><b>12</b></p>	
------------------	--

<p><b>13</b></p>	
------------------	---

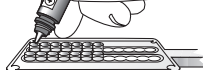
<p><b>14</b></p>	
------------------	---

<p><b>15</b></p>	
------------------	---

<p><b>16</b></p>	
------------------	---

<p><b>17</b></p>	
------------------	---

<p><b>18</b></p>	
------------------	---

<p><b>19</b></p>	
------------------	---

<p><b>20</b></p>	
------------------	---

<p><b>21</b></p>	
------------------	---

<p><b>22</b></p>	
------------------	---

<p><b>23</b></p>	
------------------	---

<p><b>24</b></p>	
------------------	---

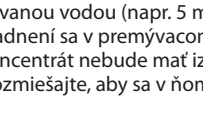
<p><b>25</b></p>	
------------------	---

<p><b>26</b></p>	
------------------	---

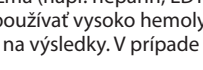
<p><b>27</b></p>	
------------------	---

<p><b>28</b></p>	
------------------	---

<p><b>29</b></p>	
------------------	---

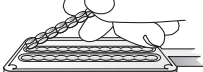
<p><b>30</b></p>	
------------------	---

<p><b>31</b></p>	
------------------	---

<p><b>32</b></p>	
------------------	---

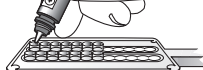
<p><b>33</b></p>	
------------------	---

<p><b>34</b></p>	
------------------	---

<p><b>35</b></p>	
------------------	---

<p><b>36</b></p>	
------------------	---

<p><b>37</b></p>	
------------------	---

<p><b>38</b></p>	
------------------	---

<p><b>39</b></p>	
------------------	---

<p><b>40</b></p>	
------------------	---

<p><b>41</b></p>	
------------------	---

<p><b>42</b></p>	
------------------	---

<p><b>43</b></p>	
------------------	---

<p><b>44</b></p>	
------------------	---

<p><b>45</b></p>	
------------------	---

<p><b>46</b></p>	
------------------	---

<p><b>47</b></p>	
------------------	---

<p><b>48</b></p>	
------------------	--

<p><b>49</b></p>	
------------------	---

<p><b>50</b></p>	
------------------	---

<p><b>51</b></p>	
------------------	---

<p><b>52</b></p> </
---------------------